

E v o l u t i s
C R E A T E U R F A B R I C A N T




MOONSTONE[®]

SURGICAL TECHNIQUE

The purpose of this surgical technique is to ensure a correct use of instruments and to make operating easier. The surgeon will adapt according to his practices .

1- Resecton of the femoral head.

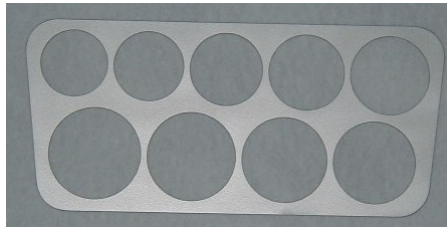

2- Measure femoral head with jauge or measuring plate



3- Bi polar cup with ring locking :

Remove the locking ring with for-
ceps, snap the cup onto the femoral
head by clip on locking ring.



**4-locked press-assembled bi
polar cup :**

Impaction with locked press.





INSTRUMENTATION FOR RING CUP

Ref.	Description
S01 006	Ring forceps

INSTRUMENTS CUP WITHOUT RING

Ref	Description
H52 033	Inlay press
H52 030	Inlay press a screw
H52 035	Mobile inlay impactor
H52 031	Pushing nozzle



INSTRUMENT SETS

Ref.	Description
H36 001	Head mesurement
H36 002	Cup impactor nozzle
H01 023	Cup impactor handle
H36 003	Measurement sheet

